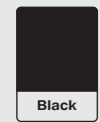
Brand: PHONAK
Ref pack: PNAK4 GAM-70_A03
Date: 18/01/2024

Software used: IN-DESIGN CC 2023
Print type: offset or digital
Print media: white paper 70 gsm offset

Colors used:

CMYK


VARNISH

Texts specifications: Regulatory texts modified by the customer (DE+ES+FR+IT+PT)

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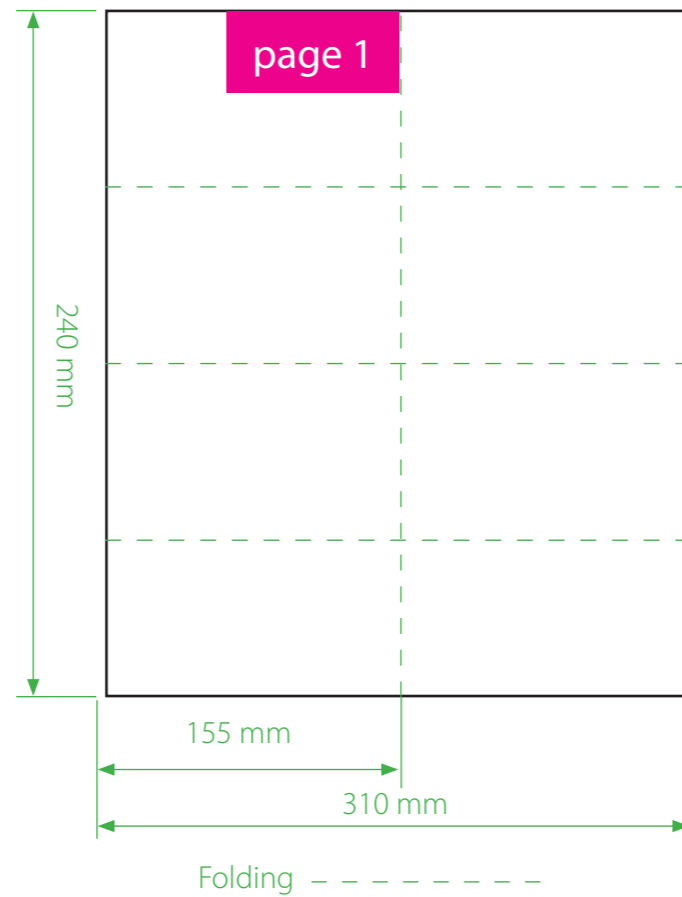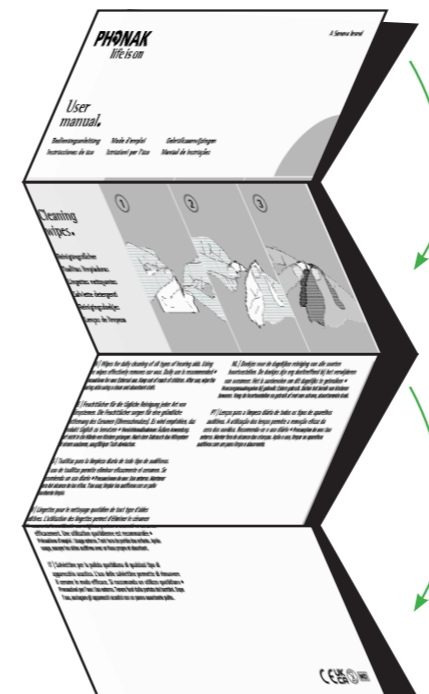
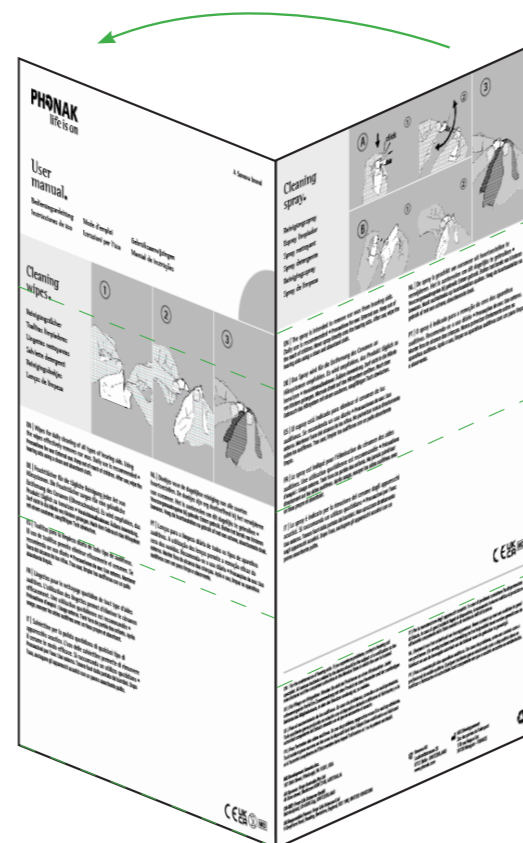
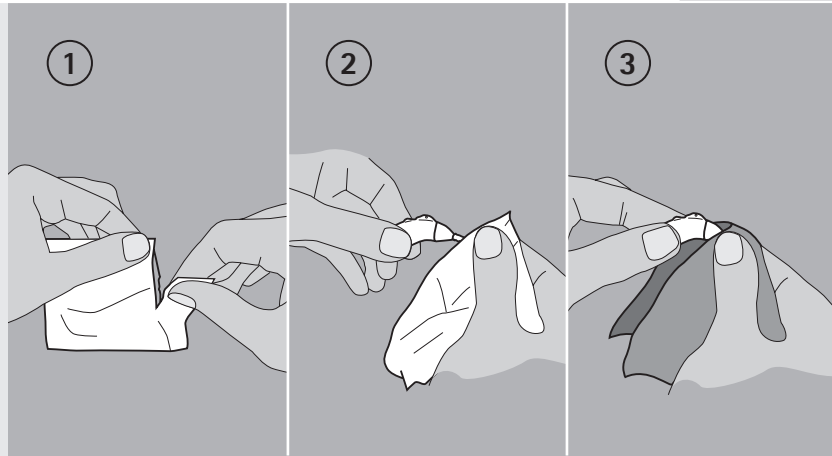
EN | Wipes for daily cleaning of all types of hearing aids. Using the wipes effectively removes ear wax. Daily use is recommended • **Precautions for use:** External use. Keep out of reach of children. After use, wipe the hearing aids using a clean and absorbent cloth.

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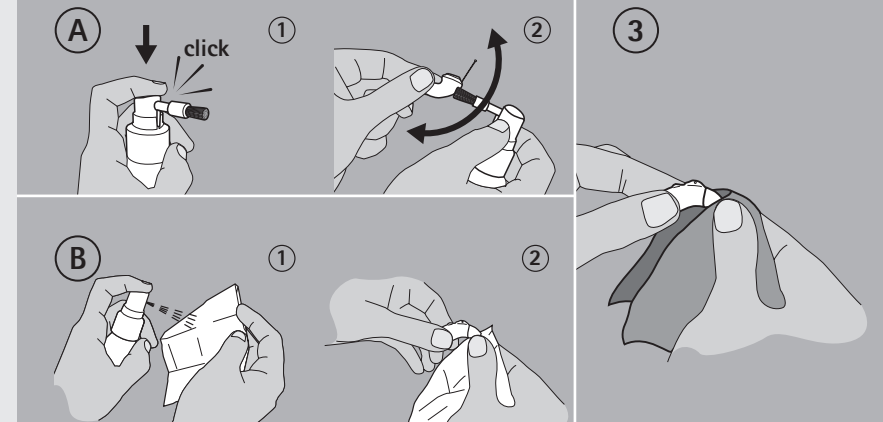
EN | The spray is intended to remove ear wax from hearing aids. Daily use is recommended • **Precautions for use:** External use. Keep out of reach of children. Never spray directly onto the hearing aids. After use, wipe the hearing aids using a clean and absorbent cloth.

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EN | For the maintenance of hearing aids. If you experience problems, contact a hearing aid specialist. All serious incidents related to the device should be the subject of a notification to the manufacturer and the competent authorities in the Member State in which the user is located.

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MG Development America Inc.
187 36th Street, Pittsburgh, PA 15201, USA

AU Sponsor: Freyr Australia Pty Ltd
46 Dora street, Blacktown NSW 2148, AUSTRALIA

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UK Responsible Person: Freyr Life Sciences Ltd
9 Greyfriars Road, Reading, Berkshire, England, RG1 1NU, UNITED KINGDOM

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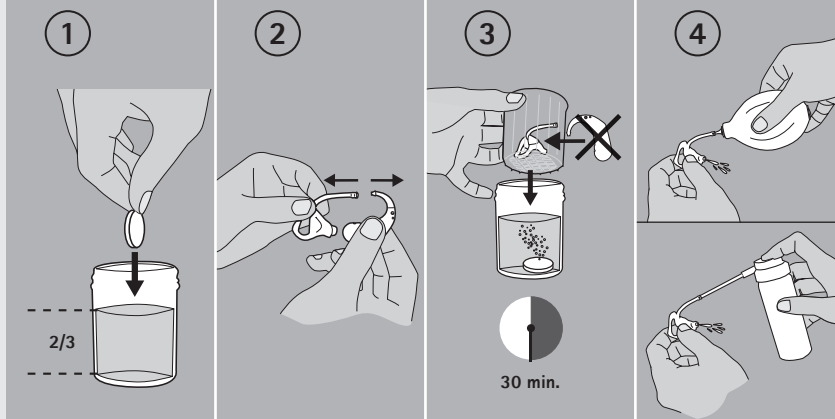
EN | Cleaning tabs for ear tips. 30 minutes are enough to effectively remove ear wax. Place the cleaning tab in the cup and fill it two thirds full of water. After cleaning, rinse the earmold with clean water and dry it. Dispose of the solution after use. Full cleaning of the ear tips is recommended once or twice per week • **Precautions for use:** External use. Single use. Keep out of reach of children. Harmful if swallowed. Causes severe skin burns and eye damage. Wash thoroughly after handling. Harmful to aquatic life – Avoid release to the environment. If medical advice is sought, keep the container or label on hand. If swallowed: Call a poison centre / a doctor. Rinse the mouth. Do not induce vomiting. If in contact with the eyes: Carefully rinse with water for several minutes.

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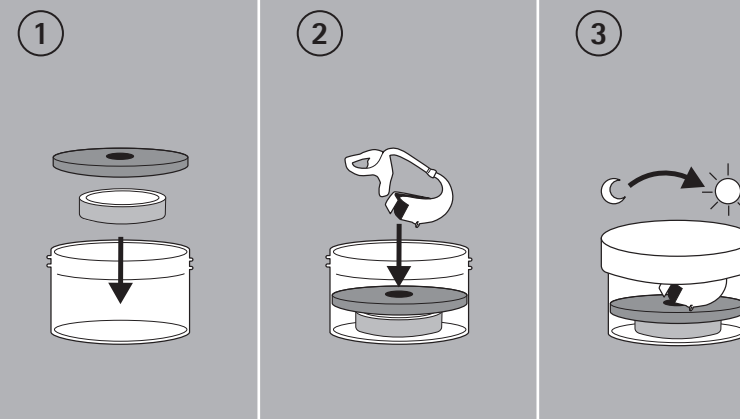
EN | Capsules to prevent damage to electronic components caused by humidity. Open the battery compartment if the hearing aid has one. Close the cup after each use. Daily use is recommended. Replace the capsule when it is almost white. The service life is about 1 month • **Precautions for use:** Do not pierce the capsule. Do not swallow. Keep out of reach of children.

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